



**REPRODUCTIVE HEALTH**

## Product Warranty

**Allwin** warrants that at the time of manufacture, the products are prepared and tested in accordance with highest standards. Due to biological differences in individuals, no product is 100% effective under all circumstances for all patients. Since we have no control over the condition under which the product is being used, the diagnosis, the method, the administration, the handling, and the storage, we do not warrant any consequences which may arise pertaining to the final use of the device. The foregoing warranty is exclusive and in lieu of all other warranties written, oral, or implied. No representative of **Allwin** may change any of the foregoing and the buyer accepts the product subject to all terms hereof.

## Design Changes

Continuous improvement is the chosen path at **Allwin** and we reserve the right to change or discontinue any product without notice.

## Return Policy


**Allwin** will accept returned goods only after issuing a Return Goods Authorisation (RGA). The return request must provide a reason for return, invoice number and order number. Credit will be issued once the request is evaluated as per company rules. We will accept a return of the stock item within 30 days of the original invoice date. We will not accept the return of a nonstock, opened, and damaged products. Returned stock items may be subject to a restocking fee.

## Storage of Products

**Allwin** recommends that products should be stored in a cool and dry place. The plastic material used may degrade following extended periods of exposure to light and temperature.

<b>OVUM PICKUP NEEDLES.....</b>	<b>3-10</b>
ACE Single Lumen Ovum Pickup Needle.....	3
ACE-FL Single Lumen Ovum Pickup Needle with Flushing Line.....	4
EAGLE Single Lumen Ovum Pickup Needle.....	5
OPICK Single Lumen Ovum Pickup Needle with Knurling.....	6
ACE-EF Single Lumen Ovum Pickup Needle with Easy Flush.....	7
ACE-M Manual Single Lumen Ovum Pickup Needle.....	8
OVUMPICK Double Lumen Ovum Pickup Needle.....	9
DUALFLOW Double Lumen Ovum Pickup Needle.....	10
<b>EMBRYO TRANSFER CATHETERS.....</b>	<b>11-19</b>
EMBRYO TRANS Embryo Transfer Catheter.....	11
BULB TRANS ULTRA Embryo Transfer Catheter - Bulb Tip.....	12
BULB TRANS STAR Embryo Transfer Catheter - Bulb Tip.....	13
BULB TRANS ULTRA PLUS Embryo Transfer Catheter - Bulb Tip.....	14
BULB TRANS STAR PLUS Embryo Transfer Catheter - Bulb Tip.....	15
PRO EMBRYO TRANS Embryo Transfer Catheter.....	16
PRO EMBRYO STAR Embryo Transfer Catheter.....	17
PRO PASS Embryo Transfer Catheter.....	18
PRO PASS STAR Embryo Transfer Catheter.....	19
<b>INTRA UTERINE INSEMINATION CATHETERS.....</b>	<b>20-25</b>
INSEMINA Intra Uterine Insemination Catheter.....	20
CURVED Intra Uterine Insemination Catheter.....	21
TRACKABLE Intra Uterine Insemination Catheter with Malleable Stylet.....	22
FLEXI Intra Uterine Insemination Catheter.....	23
INTEGRA Intra Uterine Insemination Catheter .....	24
STUMPY Intra Uterine Insemination Cannula.....	25
<b>FEMALE REPRODUCTIVE SYSTEM.....</b>	<b>26</b>

## ACE Single Lumen Ovum Pickup Needle

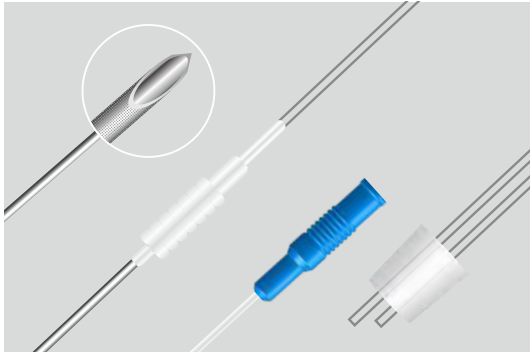

The ACE Single Lumen Ovum Pickup Needle is used for laparoscopic or ultrasound-guided transvaginal retrieval of oocytes from ovarian follicles.

- Echogenic tip
- Silicone bung for a tight fit
- Non-coring needle
- With luer lock connector
- Ergonomic handle
- 100 cm aspiration line

Tri-facet geometry  
with a echogenic tip

Ergonomic handle

Aspiration line


Silicone bung  
for tight fit

Luer lock connector

Vacuum line


Order No.	Description	Needle Gauge	Needle Length (cm)	Qty. / Box
A-OVM-SL16/35	ACE Single Lumen Ovum Pickup Needle	16	35	10
A-OVM-SL17/35	ACE Single Lumen Ovum Pickup Needle	17	35	10
A-OVM-SL18/35	ACE Single Lumen Ovum Pickup Needle	18	35	10

\*All products may not be available in all countries. Please contact our sales department for details.

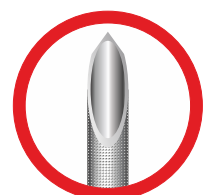


Echogenic tip

## ACE-FL Single Lumen Ovum Pickup Needle with Flushing Line

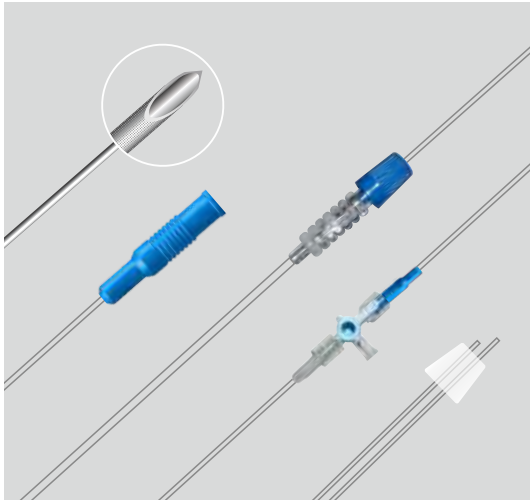


The ACE-FL Single Lumen Ovum Pickup Needle with Flushing Line is used for laparoscopic or ultrasound-guided transvaginal retrieval of oocytes from ovarian follicles.

- Aspiration and flushing line is made possible through a single channel
- Echogenic tip
- Silicone bung for a tight fit
- Non-coring needle
- With luer lock connector
- Ergonomic handle
- 100 cm aspiration line
- Multiple operation functionality by using stopcock valve

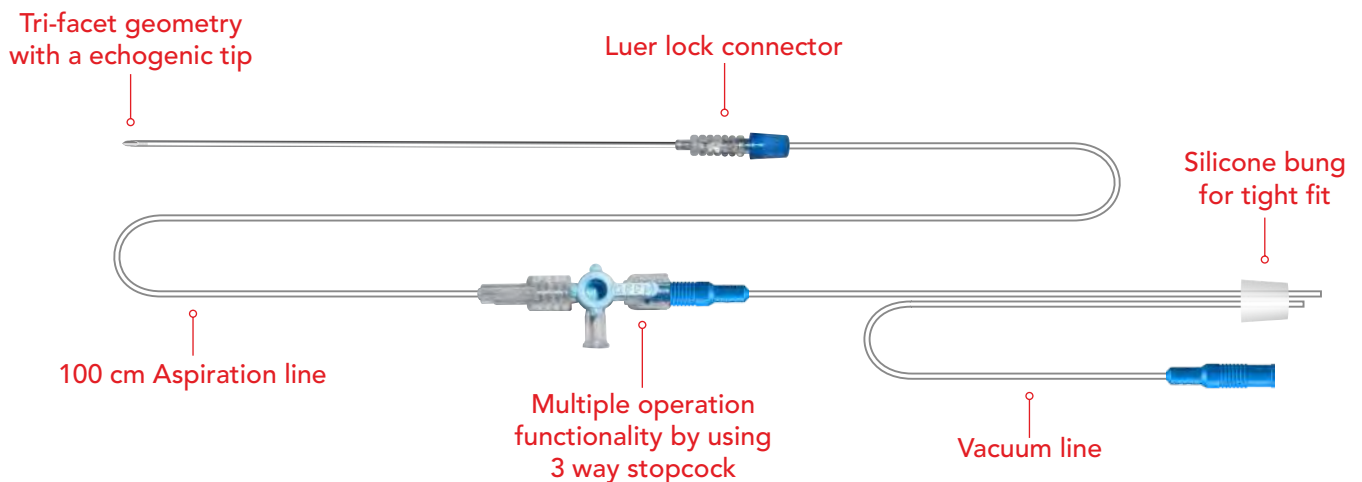


Order No.	Description	Needle Gauge	Needle Length (cm)	Qty. / Box
A-OVM-SL16/35-FL	ACE-FL Single Lumen Ovum Pickup Needle with Flushing Line	16	35	10
A-OVM-SL17/35-FL	ACE-FL Single Lumen Ovum Pickup Needle with Flushing Line	17	35	10
A-OVM-SL18/35-FL	ACE-FL Single Lumen Ovum Pickup Needle with Flushing Line	18	35	10

\*All products may not be available in all countries. Please contact our sales department for details.

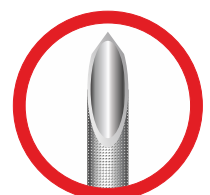

Echogenic tip

## EAGLE Single Lumen Ovum Pickup Needle

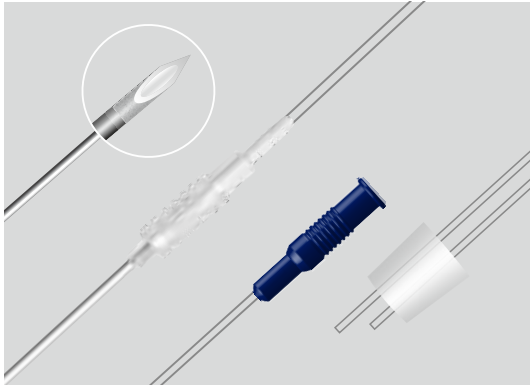

The EAGLE Single Lumen Ovum Pickup Needle is used for laparoscopic or ultrasound-guided transvaginal retrieval of oocytes from ovarian follicles.

- Sharp tip for easier penetration
- Echogenic tip
- Silicone bung for a tight fit
- Non-coring needle
- With luer lock connector
- Ergonomic handle
- 100 cm aspiration line

Tri-facet geometry  
with a echogenic tip

Ergonomic handle

Aspiration line

Silicone bung  
for tight fit

Luer lock connector

Vacuum line


Order No.	Description	Needle Gauge	Needle Length (cm)	Qty. / Box
E-OVM-SL17/35	EAGLE Single Lumen Ovum Pickup Needle	17	35	10

\*All products may not be available in all countries. Please contact our sales department for details.



Echogenic tip


## OPICK Single Lumen Ovum Pickup Needle with Knurling

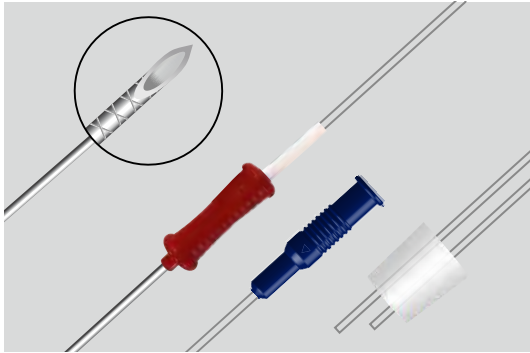

The OPICK Single Lumen Ovum Pickup Needle with Knurling is used for laparoscopic or ultrasound - guided transvaginal retrieval of oocytes from ovarian follicles.


- Echogenic tip with knurling
- Silicone bung for a tight fit
- Non-coring needle
- With luer lock connector
- Ergonomic handle
- 100 cm aspiration line

Tri-facet geometry  
with a echogenic tip

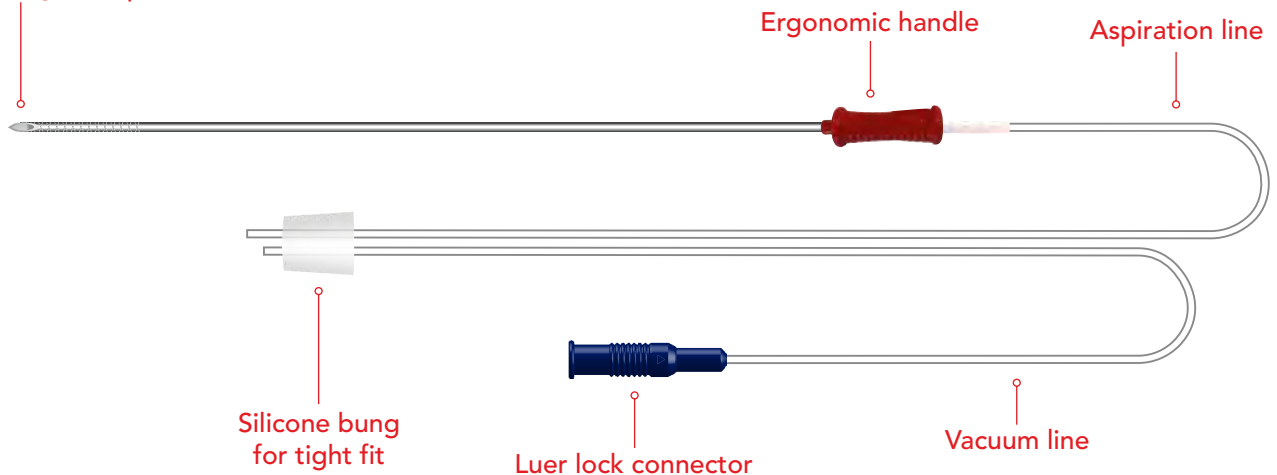


Order No.	Description	Needle Gauge	Needle Length (cm)	Qty. / Box
OPN-SL17/35-K	OPICK Single Lumen Ovum Pickup Needle with Knurling	17	35	10

\*All products may not be available in all countries. Please contact our sales department for details.

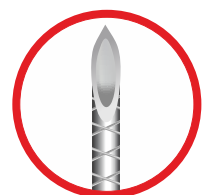


Echogenic tip  
with knurling

## ACE-EF Single Lumen Ovum Pickup Needle with Easy Flush

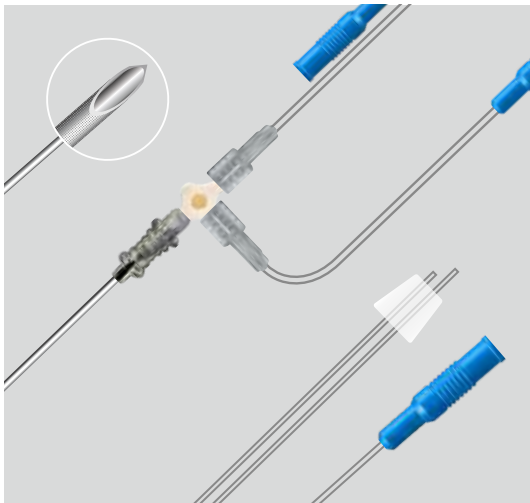

The ACE-EF Single Lumen Ovum Pickup Needle with Easy Flush is used for laparoscopic or ultrasound - guided transvaginal retrieval of oocytes from ovarian follicles.

- Aspiration and flushing line is made possible through a single channel
- Echogenic tip
- Silicone bung for a tight fit
- Non-coring needle
- With luer lock connector
- Dismantling of luer lock connector
- Ergonomic handle
- 100 cm aspiration line

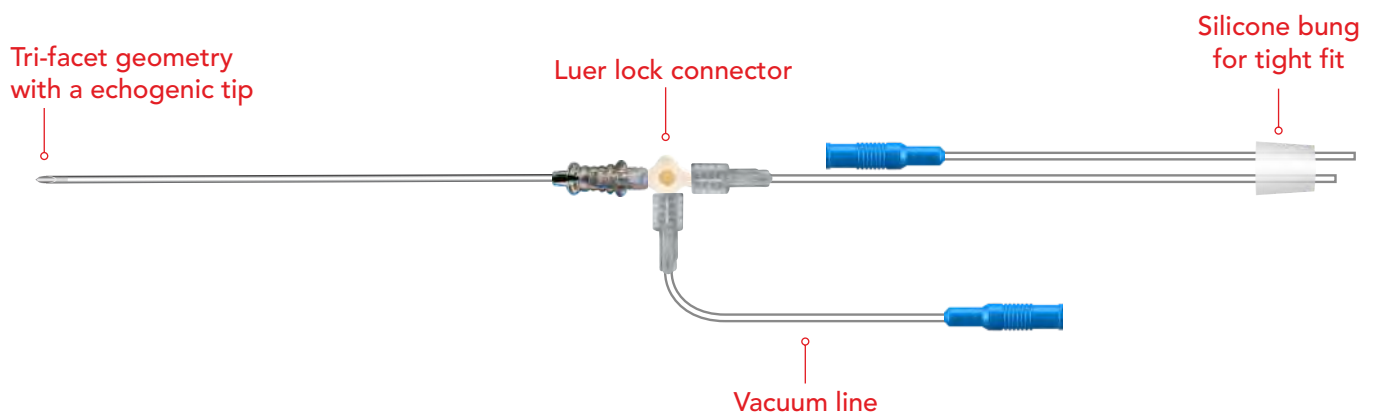

Order No.	Description	Needle Gauge	Needle Length (cm)	Qty. / Box
A-OVM-SL16/35-EF	ACE-EF Single Lumen Ovum Pickup Needle with Easy Flush	16	35	10
A-OVM-SL17/35-EF	ACE-EF Single Lumen Ovum Pickup Needle with Easy Flush	17	35	10
A-OVM-SL18/35-EF	ACE-EF Single Lumen Ovum Pickup Needle with Easy Flush	18	35	10

\*All products may not be available in all countries. Please contact our sales department for details.




Dismantling of  
luer lock connector



Echogenic tip



## ACE-M Manual Single Lumen Ovum Pickup Needle

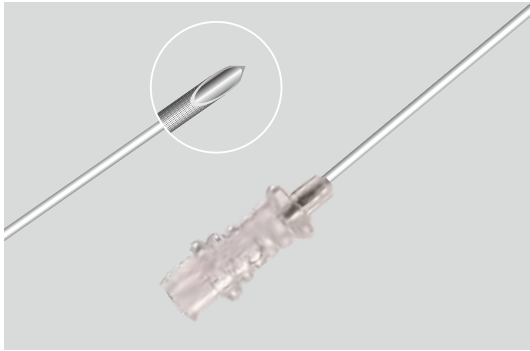

The ACE-M Manual Single Lumen Ovum Pickup Needle is used for laparoscopic or ultrasound-guided transvaginal retrieval of oocytes from follicles.

- When used with ultrasonic equipment, echo tip enhances visualization
- Handle designed for greater control during ovum retrieval
- Luer lock connector for connecting a syringe
- The amount of suction applied is manually controlled


Tri-facet geometry  
with a echogenic tip



Ergonomic  
handle


Order No.	Description	Needle Gauge	Needle Length (cm)	Qty. / Box
A-OVM-SL16/35-M	ACE-M Manual Single Lumen Ovum Pickup Needle	16	35	10
A-OVM-SL17/35-M	ACE-M Manual Single Lumen Ovum Pickup Needle	17	35	10
A-OVM-SL18/35-M	ACE-M Manual Single Lumen Ovum Pickup Needle	18	35	10

\*All products may not be available in all countries. Please contact our sales department for details.

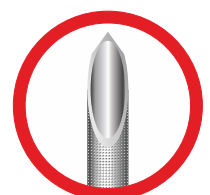


Echogenic tip

## OVUMPICK Double Lumen Ovum Pickup Needle

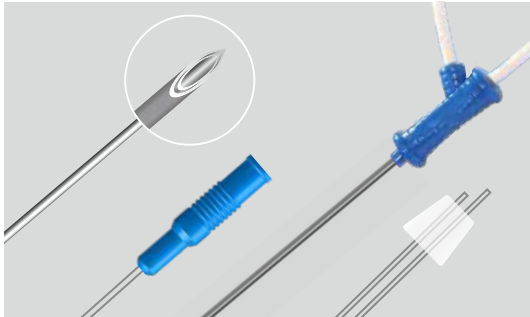

The OVUMPICK Double Lumen Ovum Pickup Needle is used for laparoscopic or ultrasound-guided transvaginal retrieval of oocytes from ovarian follicles.

- Highly echogenic tip
- Silicone bung for a tight fit
- Non-coring needle
- With Luer lock connector
- The outer lumen allows flushing of follicles

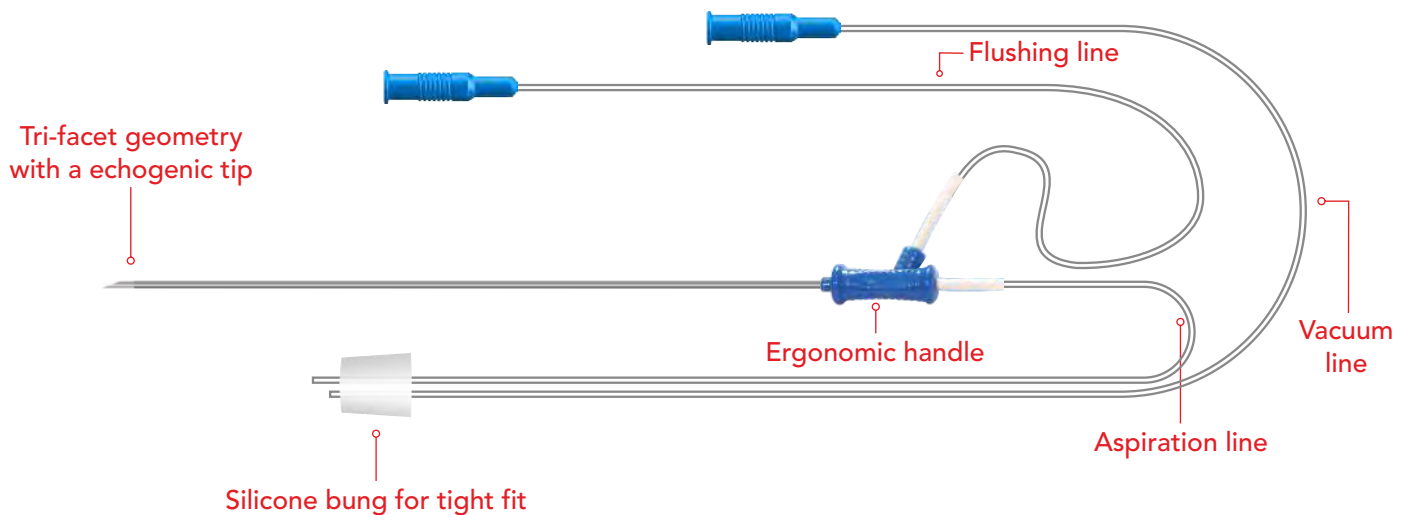


Order No.	Description	Needle Gauge	Needle Length (cm)	Qty. / Box
OVM-DL16/35	OVUMPICK Double Lumen Ovum Pickup Needle	16	35	10
OVM-DL17/35	OVUMPICK Double Lumen Ovum Pickup Needle	17	35	10

\*All products may not be available in all countries. Please contact our sales department for details.




Echogenic tip

## DUALFLOW Double Lumen Ovum Pickup Needle

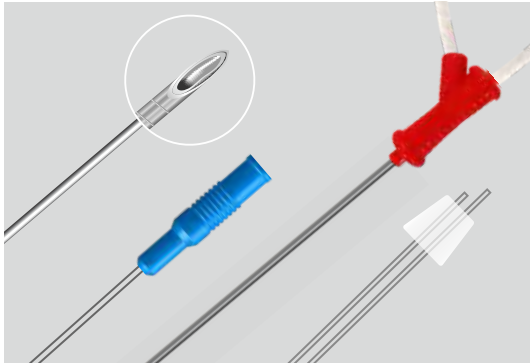

The DUALFLOW OVUMPICK Double Lumen Ovum Pickup Needle is used for laparoscopic or ultrasound-guided transvaginal retrieval of oocytes from ovarian follicles.

- Highly echogenic tip
- Silicone bung for a tight fit
- Non-coring needle
- With Luer lock connector
- The outer lumen allows flushing of follicles

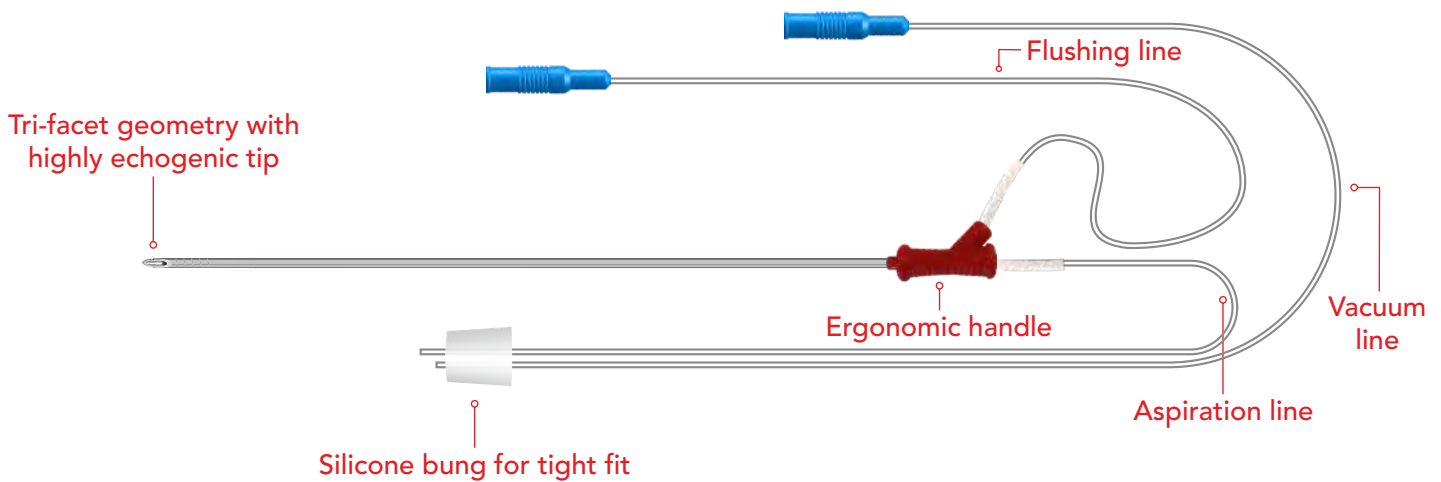


Order No.	Description	Needle Gauge	Needle Length (cm)	Qty. / Box
DUF-DL17/35	DUALFLOW Double Lumen Ovum Pickup Needle	17	35	10

\*All products may not be available in all countries. Please contact our sales department for details.




Echogenic tip

## EMBRYO TRANS Embryo Transfer Catheter

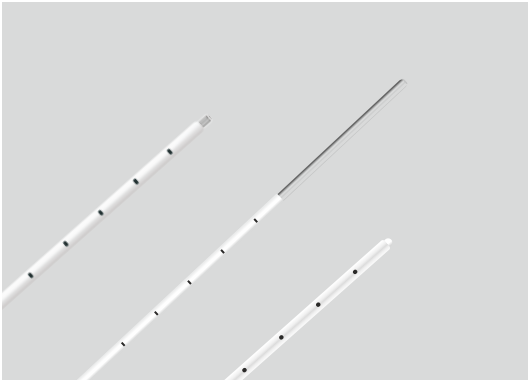

The EMBRYO TRANS Embryo Transfer Catheter is used to introduce in-vitro fertilized (IVF) embryos into the uterine cavity.

- A soft inner catheter with smooth tip
- Marking on inner catheter and outer sheath
- Shapeable outer sheath
- Stylet for outer sheath available separately

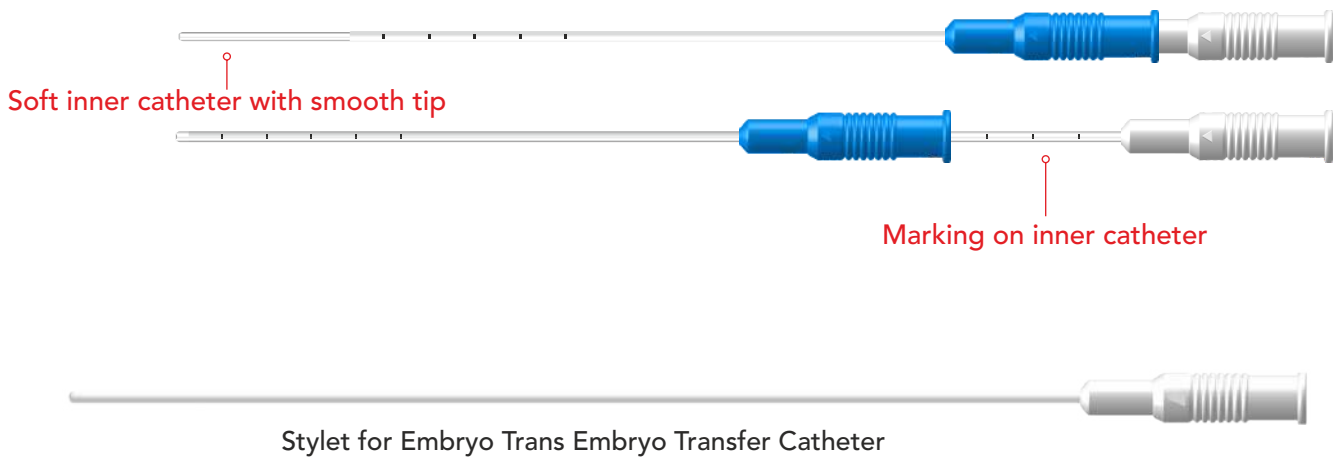


Order No.	Description	Catheter Length (cm)	Qty. / Box
EB-ETC23	EMBRYO TRANS Embryo Transfer Catheter	23	10
EB-ETC18	EMBRYO TRANS Embryo Transfer Catheter	18	10
STY23	Stylet for Embryo Trans Embryo Transfer Catheter	23	10
STY18	Stylet for Embryo Trans Embryo Transfer Catheter	18	10
STY23-OS	Stylet for Introducing Catheter 23cm with Outer Sheath	23	10
STY18-OS	Stylet for Introducing Catheter 18cm with Outer Sheath	18	10

\*All products may not be available in all countries. Please contact our sales department for details.




Soft inner catheter

## BULB TRANS ULTRA Embryo Transfer Catheter - Bulb Tip

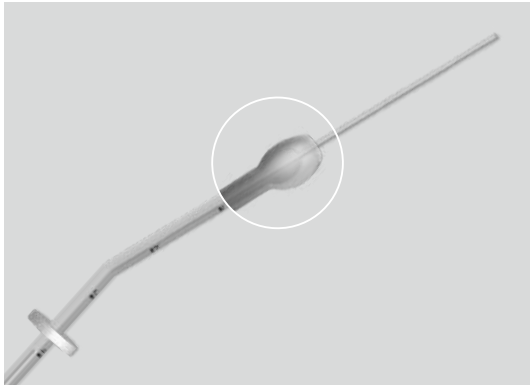

The BULB TRANS ULTRA Embryo Transfer Catheter - Bulb Tip is used to introduce in-vitro fertilized (IVF) embryos into the uterine cavity.

- Firm outer sheath
- A supported soft inner catheter with a smooth tip
- Marking on inner steel pipe and outer sheath
- Bulb shaped tip of the outer sheath
- Silicone depth positioner for precise placement
- The bulb-shaped angled outer sheath eases passage through the cervix

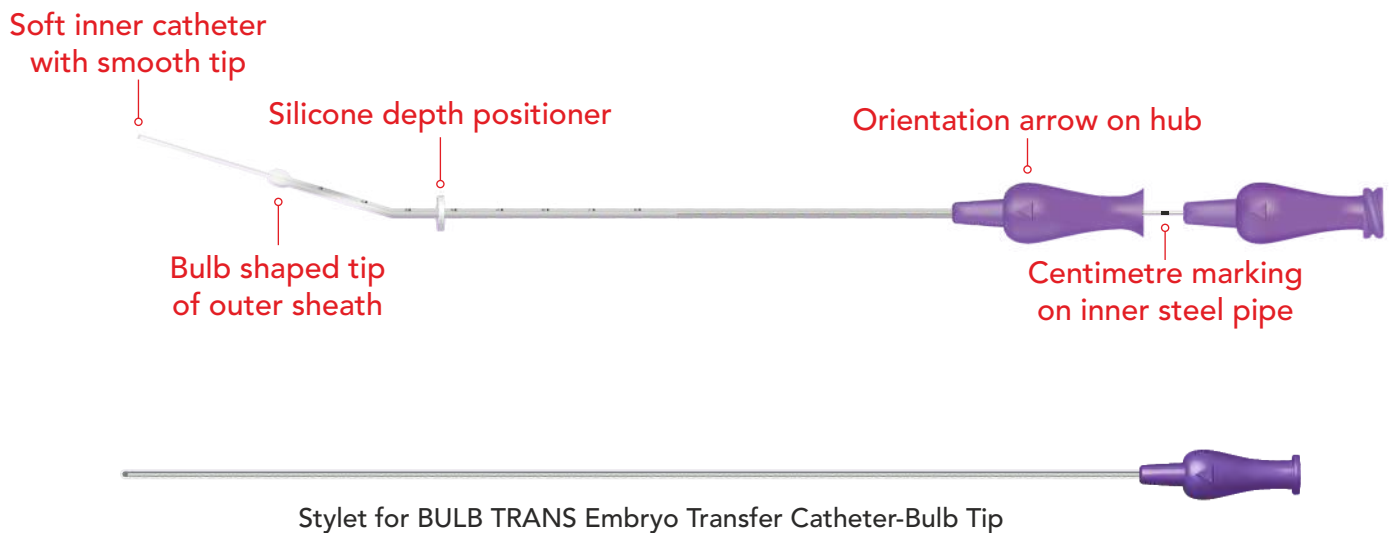


Order No.	Description	Qty. / Box
BT-ETC-U	BULB TRANS ULTRA Embryo Transfer Catheter - Bulb Tip	10
STY-BU	Stylet for BULB TRANS ULTRA Embryo Transfer Catheter - Bulb Tip	10

\*All products may not be available in all countries. Please contact our sales department for details.




Soft inner catheter with smooth tip

## BULB TRANS STAR Embryo Transfer Catheter - Bulb Tip

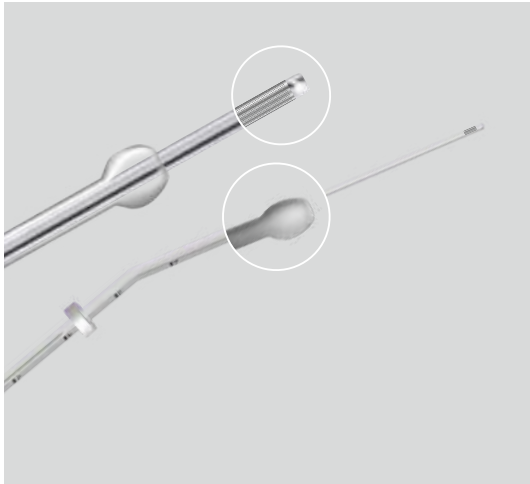



The BULB TRANS STAR Embryo Transfer Catheter - Bulb Tip is used to introduce in-vitro fertilized (IVF) embryos into the uterine cavity.

- Echogenic band on the inner catheter to provide visibility under ultrasound
- Firm outer sheath
- A supported soft inner catheter with a smooth tip
- Marking on inner steel pipe and outer sheath
- Bulb shaped angled tip of the outer sheath
- Silicone depth positioner for precise placement
- The catheter is curved to facilitate insertion

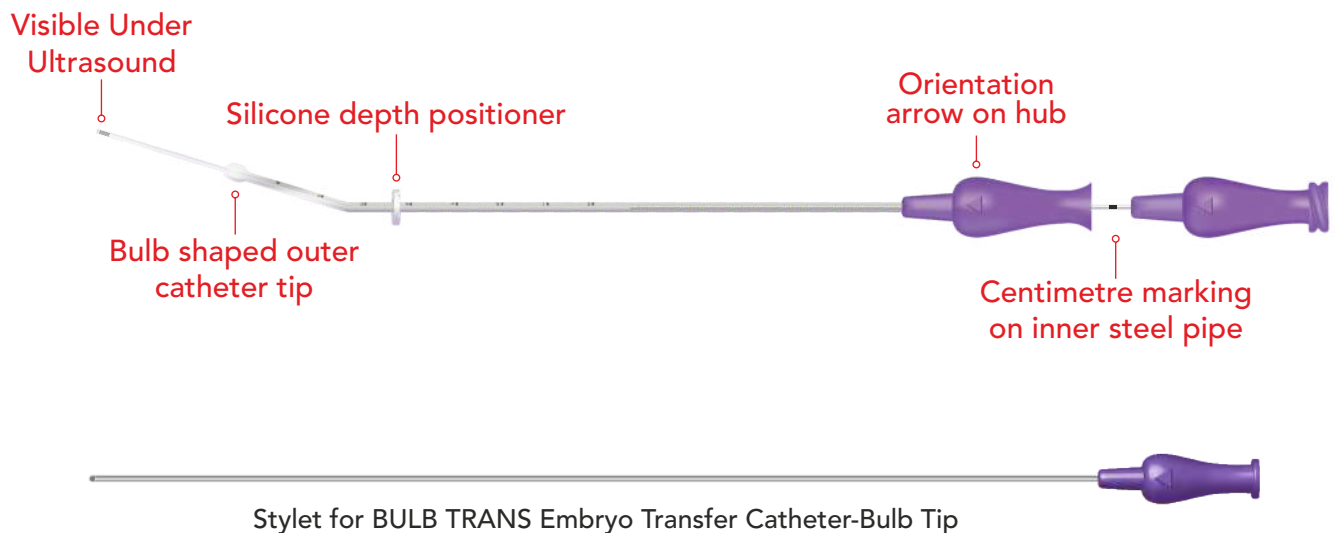


Order No.	Description	Qty. / Box
BT-ETC-U-S	BULB TRANS STAR Embryo Transfer Catheter - Bulb Tip	10
STY-BU	Stylet for BULB TRANS STAR Embryo Transfer Catheter - Bulb Tip	10

\*All products may not be available in all countries. Please contact our sales department for details.

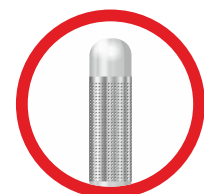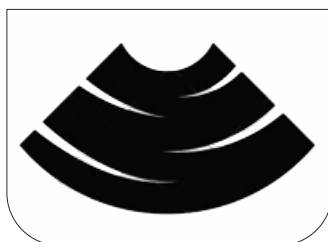


Dimpled echogenic metal band with soft inner catheter

## BULB TRANS ULTRA PLUS Embryo Transfer Catheter - Bulb Tip

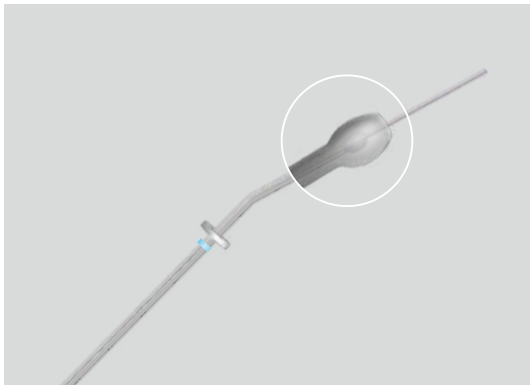

The BULB TRANS ULTRA PLUS Embryo Transfer Catheter - Bulb Tip is used to introduce in-vitro fertilized (IVF) embryos into the uterine cavity.

- Firm outer sheath
- A supported soft inner catheter with a smooth tip
- Marking on inner steel pipe
- Bulb shaped tip of the outer sheath
- Silicone depth positioner for precise placement
- The bulb-shaped angled outer sheath eases passage through the cervix

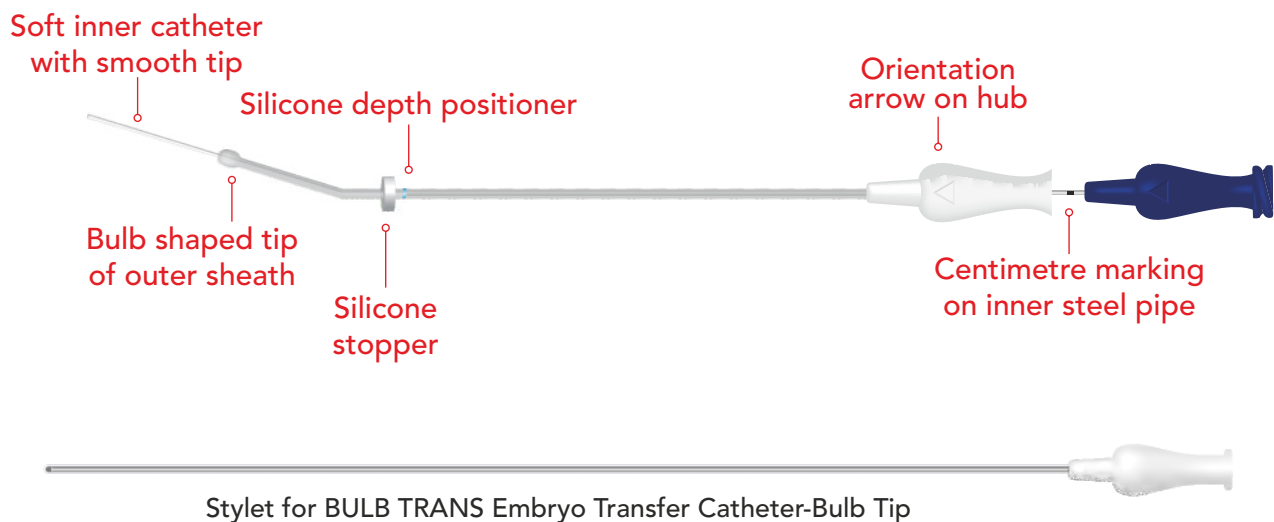


Order No.	Description	Qty. / Box
BT-ETC-U-P	BULB TRANS ULTRA PLUS Embryo Transfer Catheter - Bulb Tip	10
STY-BU	Stylet for BULB TRANS ULTRA PLUS Embryo Transfer Catheter - Bulb Tip	10

\*All products may not be available in all countries. Please contact our sales department for details.




Soft inner catheter

## BULB TRANS STAR PLUS Embryo Transfer Catheter - Bulb Tip

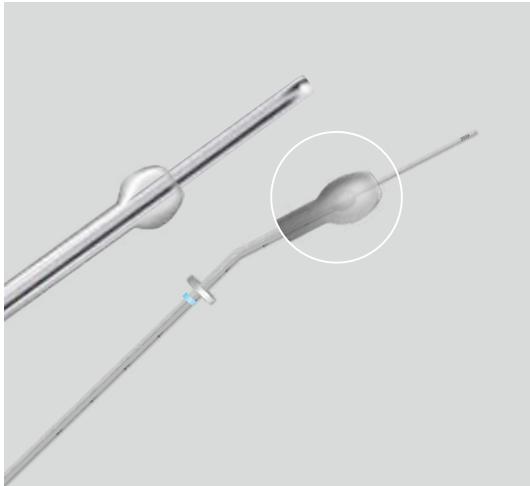


The BULB TRANS STAR PLUS Embryo Transfer Catheter - Bulb Tip is used to introduce in-vitro fertilized (IVF) embryos into the uterine cavity.

- Firm outer sheath
- A supported soft inner catheter with a smooth tip
- Marking on inner steel pipe
- Bulb shaped angled tip of the outer sheath
- Silicone depth positioner for precise placement
- The bulb-shaped angled outer sheath eases passage through the cervix
- Echogenic band on the inner catheter to improve the visibility under ultrasound

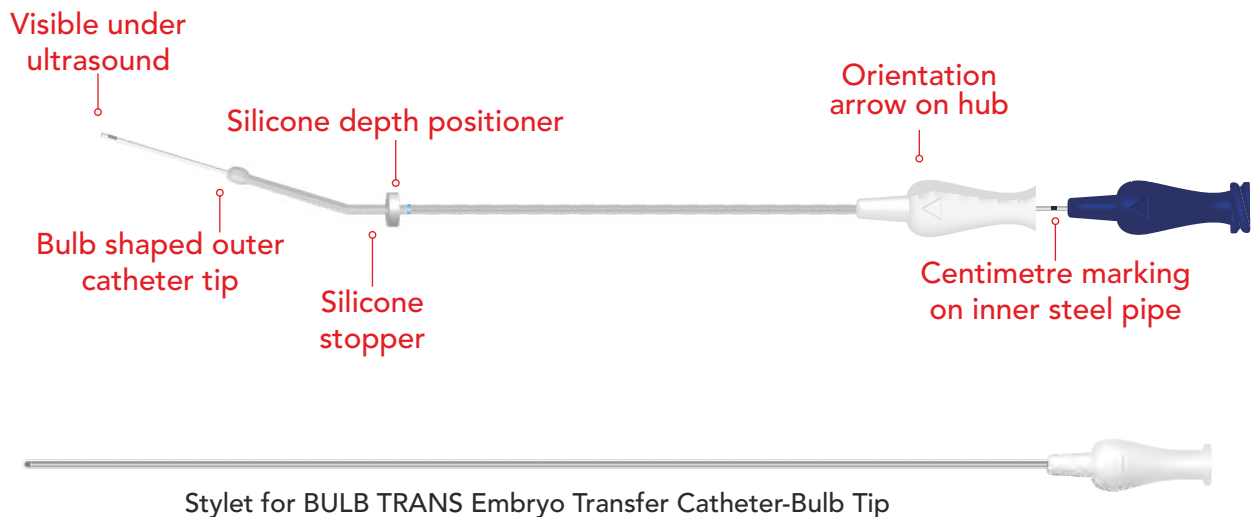

Order No.	Description	Qty. / Box
BT-ETC-U-S-P	BULB TRANS STAR PLUS Embryo Transfer Catheter - Bulb Tip	10
STY-BU	Stylet for BULB TRANS STAR PLUS Embryo Transfer Catheter - Bulb Tip	10

\*All products may not be available in all countries. Please contact our sales department for details.

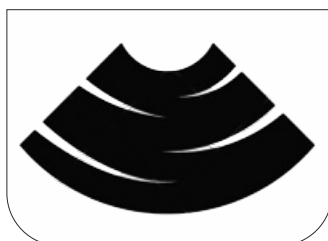

Premium bulb tip catheter with dimpled echogenic metal band




## PRO EMBRYO TRANS Embryo Transfer Catheter




The PRO EMBRYO TRANS Embryo Transfer Catheter is used to introduce in-vitro fertilized (IVF) embryos into the uterine cavity.

- A soft inner catheter & smooth tip
- Marking on inner steel pipe & outer sheath
- Flexible catheter
- Silicone depth positioner for precise placement
- Orientation arrow on the hub
- Angled outer sheath provides easy passage through the cervical canal
- Available in compatible stylet or soft obturator

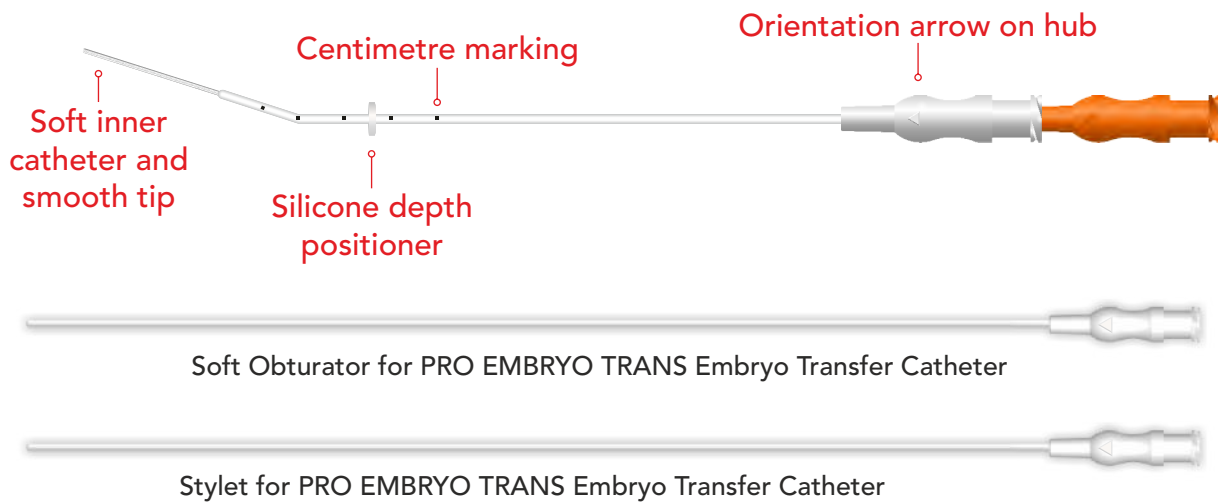


Order No.	Description	Qty. / Box
PEB-ETC	PRO EMBRYO TRANS Embryo Transfer Catheter	10
PEB-ETC-SB	PRO EMBRYO TRANS Embryo Transfer Catheter with Soft Obturator	10
PEB-ETC-STY	PRO EMBRYO TRANS Embryo Transfer Catheter with Stylet	10
SB-PE	Soft Obturator for Pro Embryo Trans / Pro Embryo Star Embryo Transfer Catheter	10
STY-PE	Stylet for Pro Embryo Trans / Pro Embryo Star Embryo Transfer Catheter	10

\*All products may not be available in all countries. Please contact our sales department for details.

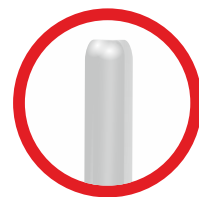

Supported inner catheter  
with smooth tip

## PRO EMBRYO STAR Embryo Transfer Catheter

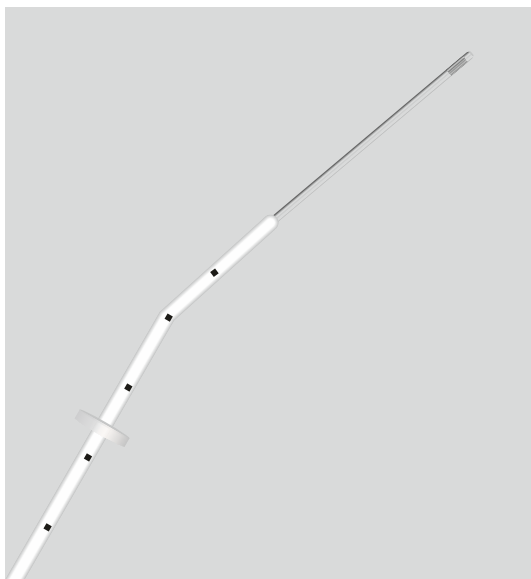

The PRO EMBRYO STAR Embryo Transfer Catheter is used to introduce in-vitro fertilized (IVF) embryos into the uterine cavity.

- Echogenic band on the inner catheter to provide visibility under ultrasound
- Marking on inner steel pipe & outer sheath
- Flexible catheter
- Silicone depth positioner for precise placement
- Orientation arrow on the hub
- Angled outer sheath provides easy passage through the cervical canal
- Available in compatible stylet or soft obturator




Soft Obturator for PRO EMBRYO STAR Embryo Transfer Catheter




Stylet for PRO EMBRYO STAR Embryo Transfer Catheter

Order No.	Description	Qty. / Box
PEB-ETC-S	PRO EMBRYO STAR Embryo Transfer Catheter	10
PEB-ETC-S-SB	PRO EMBRYO STAR Embryo Transfer Catheter with Soft Obturator	10
PEB-ETC-S-STY	PRO EMBRYO STAR Embryo Transfer Catheter with Stylet	10
SB-PE	Soft Obturator for Pro Embryo Trans / Pro Embryo Star Embryo Transfer Catheter	10
STY-PE	Stylet for Pro Embryo Trans / Pro Embryo Star Embryo Transfer Catheter	10

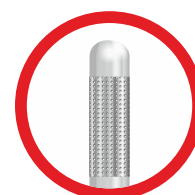


Echogenic metal band

## PRO PASS Embryo Transfer Catheter

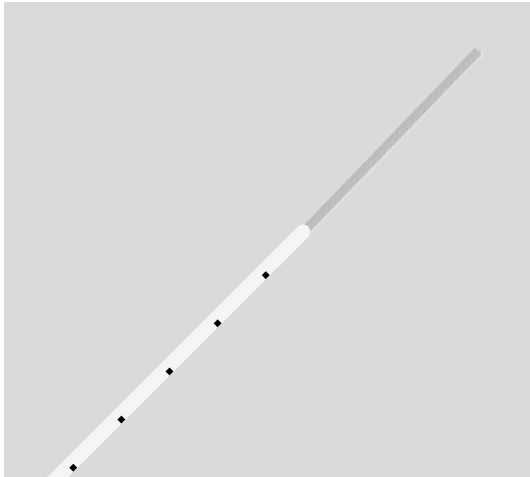

The PRO PASS Embryo Transfer Catheter is used to introduce in-vitro fertilized (IVF) embryos into the uterine cavity.

- A soft inner catheter & smooth tip
- Marking on inner steel pipe & outer sheath
- Flexible catheter
- Orientation arrow on the hub
- Available in compatible stylet or soft obturator

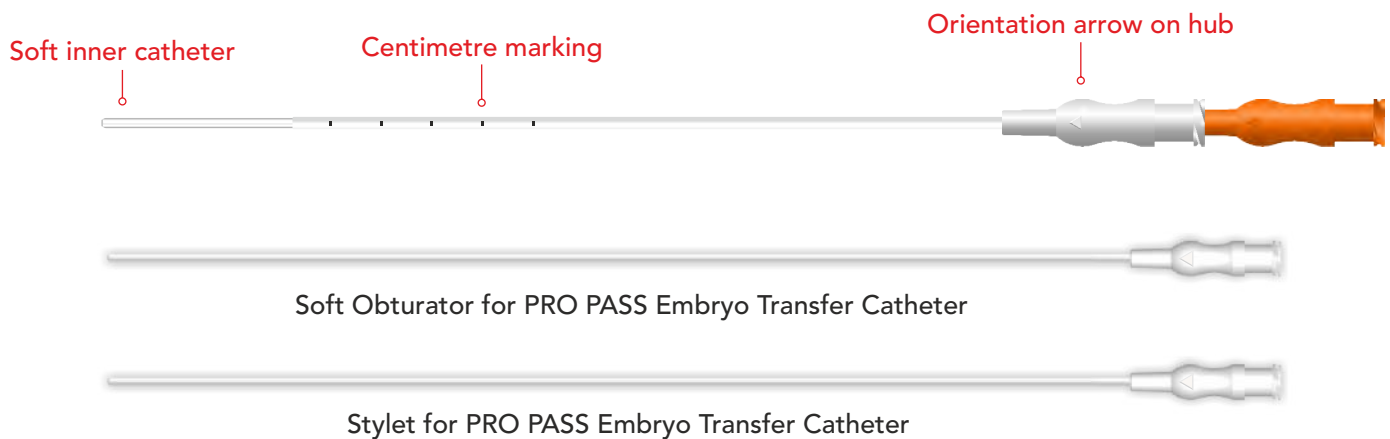


Order No.	Description	Qty. / Box
PP-ETC	PRO PASS Embryo Transfer Catheter	10
PP-ETC-SB	PRO PASS Embryo Transfer Catheter with Soft Obturator	10
PP-ETC-STY	PRO PASS Embryo Transfer Catheter with Stylet	10
SB-PE	Soft Obturator for Pro Pass / Pro Pass Star Embryo Transfer Catheter	10
STY-PE	Stylet for Pro Pass / Pro Pass Star Embryo Transfer Catheter	10

\*All products may not be available in all countries. Please contact our sales department for details.



Soft inner catheter

## PRO PASS STAR Embryo Transfer Catheter

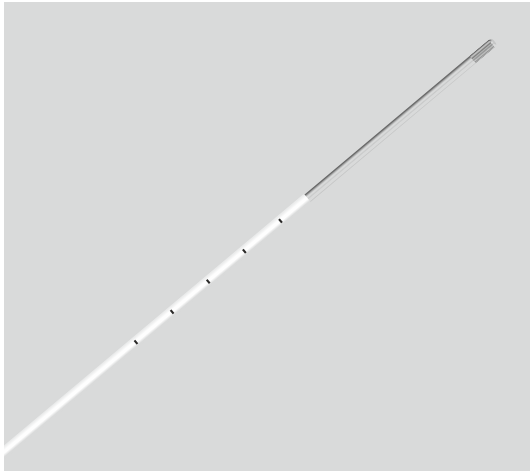

The PRO PASS STAR Embryo Transfer Catheter is used to introduce in-vitro fertilized (IVF) embryos into the uterine cavity.

- A soft inner catheter & smooth tip
- Marking on inner steel pipe & outer sheath
- Flexible catheter
- Orientation arrow on the hub
- Available in compatible stylet or soft obturator

Echogenic  
metal band

Centimetre marking

Orientation arrow on hub




Soft Obturator for PRO PASS STAR Embryo Transfer Catheter

Stylet for PRO PASS STAR Embryo Transfer Catheter


Order No.	Description	Qty. / Box
PP-ETC-S	PRO PASS STAR Embryo Transfer Catheter	10
PP-ETC-S-SB	PRO PASS STAR Embryo Transfer Catheter with Soft Obturator	10
PP-ETC-S-STY	PRO PASS STAR Embryo Transfer Catheter with Stylet	10
SB-PE	Soft Obturator for Pro Pass / Pro Pass Star Embryo Transfer Catheter	10
STY-PE	Stylet for Pro Pass / Pro Pass Star Embryo Transfer Catheter	10

\*All products may not be available in all countries. Please contact our sales department for details.

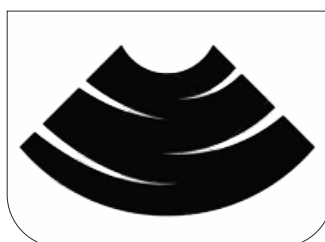


Echogenic metal band

## INSEMINA Intra Uterine Insemination Catheter

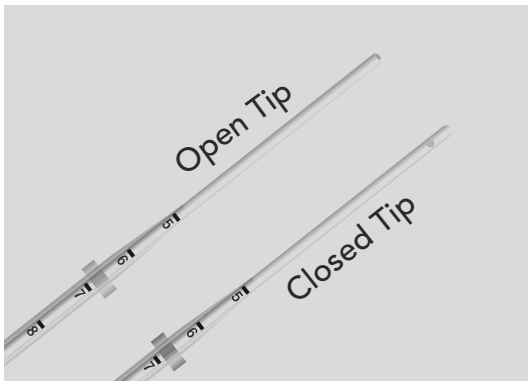


The INSEMINA Intra Uterine Insemination Catheter (Open Tip & Close Tip) is intended for the introduction of washed spermatozoa into the uterine cavity.

- A soft uterine catheter with firm support
- Integrally molded hub
- Silicone depth positioner for precise placement
- Open tip as well as closed tip (with two lateral holes) catheters available

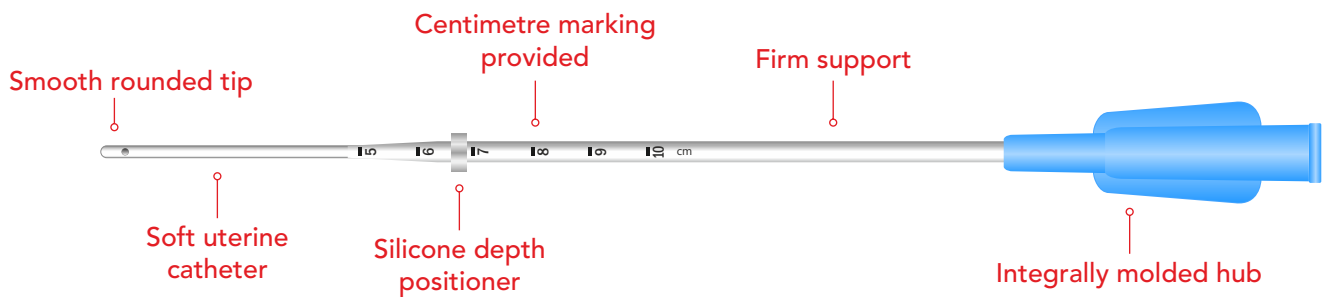


Order No.	Description	Qty. / Box
I-IUI	INSEMINA Intra Uterine Insemination Catheter	25
I-IUI-O	INSEMINA Intra Uterine Insemination Catheter (Open Tip)	25

\*All products may not be available in all countries. Please contact our sales department for details.

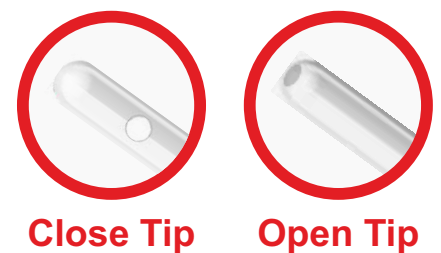


## CURVED Intra Uterine Insemination Catheter

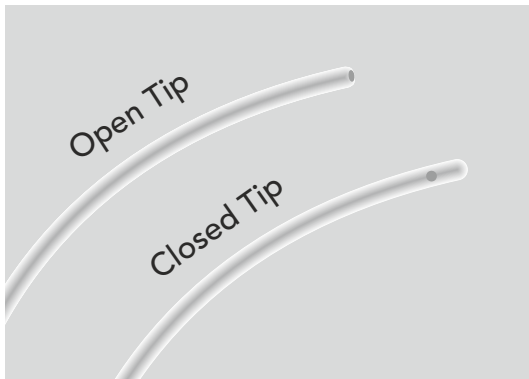


The CURVED Intra Uterine Insemination Catheter (Open Tip & Close Tip) is intended for the introduction of washed spermatozoa into the uterine cavity.

- A soft uterine catheter with firm support
- Integrally molded hub
- Silicone depth positioner for precise placement
- Curve shaped catheter
- Open tip as well as closed tip (with two lateral holes) catheters available

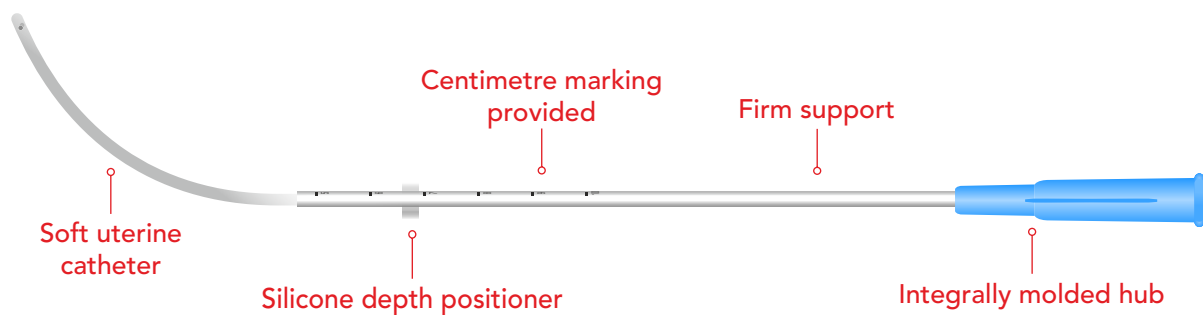


Order No.	Description	Qty. / Box
I-IUI-C	CURVED Intra Uterine Insemination Catheter	25
I-IUI-C-O	CURVED Intra Uterine Insemination Catheter (Open Tip)	25

\*All products may not be available in all countries. Please contact our sales department for details.

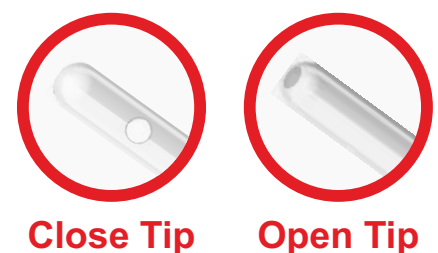


## TRACKABLE Intra Uterine Insemination Catheter with Malleable Stylet

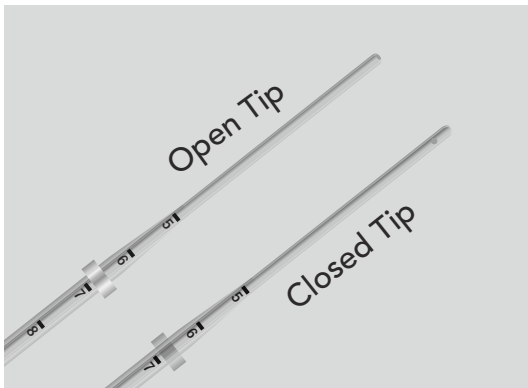


The TRACKABLE Intra Uterine Insemination Catheter with Malleable Stylet (Open Tip & Close Tip) is intended for the introduction of washed spermatozoa into the uterine cavity.

- A soft uterine catheter with firm support
- Integrally molded hub
- Silicone depth positioner for precise placement
- Malleable stylet to aid catheter insertion
- Open tip as well as closed tip (with two lateral holes) catheters available

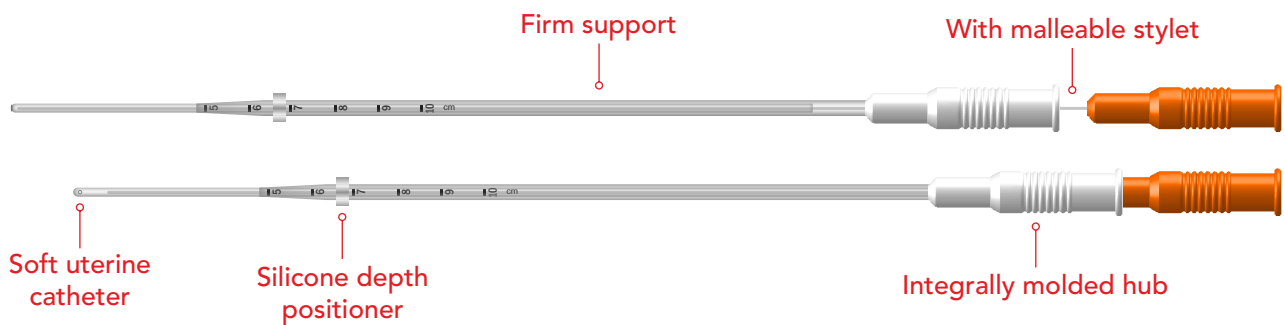


Order No.	Description	Qty. / Box
T-IUI	TRACKABLE Intra Uterine Insemination Catheter with Malleable Stylet	25
T-IUI-O	TRACKABLE Intra Uterine Insemination Catheter with Malleable Stylet (Open Tip)	25

\*All products may not be available in all countries. Please contact our sales department for details.

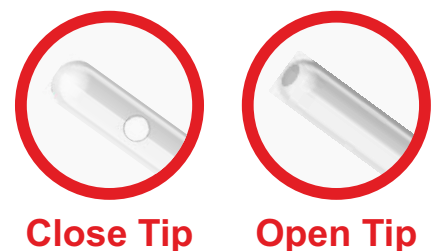

## FLEXI Intra Uterine Insemination Catheter

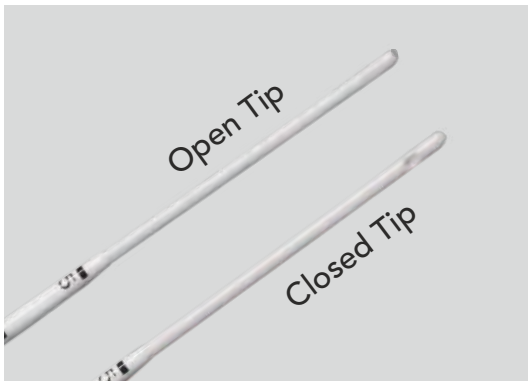

The FLEXI Intra Uterine Insemination Catheter (Open Tip & Close Tip) is intended for the introduction of washed spermatozoa into the uterine cavity.

- Shapeable outer sheath for easy insertion
- Centimetre marking on a catheter for precise placement
- Stylet available separately
- Open tip as well as closed tip (with two lateral holes) catheters available

Centimetre marking provided



Soft uterine catheter

Shapeable  
outer sheath


Firm  
support



Stylet

Order No.	Description	Qty. / Box
F-IUI	FLEXI Intra Uterine Insemination Catheter	25
F-IUI-O	FLEXI Intra Uterine Insemination Catheter (Open Tip)	25
STY18	Stylet for FLEXI IUI Intra Uterine Insemination Catheter	25

\*All products may not be available in all countries. Please contact our sales department for details.

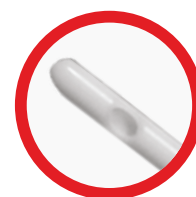


Close Tip



Open Tip




## INTEGRA Intra Uterine Insemination Catheter

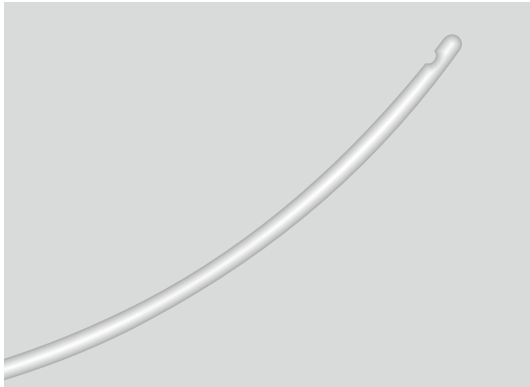

The INTEGRA Intra Uterine Insemination Catheter (Close Tip) is intended for the introduction of washed spermatozoa into the uterine cavity.

- Placement becomes easier with the distal tip's soft, flexible curve.
- The stainless steel inner cannula is malleable.
- Luer lock connector.
- Silicone stopper for adjustable positioning.

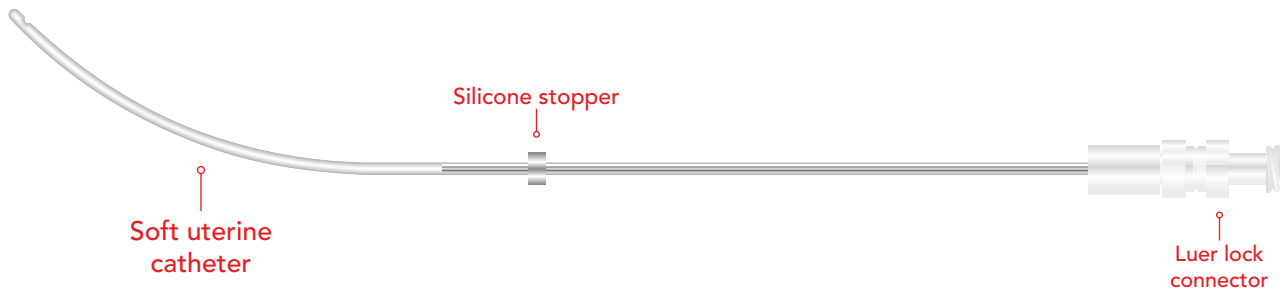

Order No.	Description	Qty. / Box
IG-IUI	INTEGRA Intra Uterine Insemination Catheter	25

\*All products may not be available in all countries. Please contact our sales department for details.



**Side Hole**

## STUMPY Intra Uterine Insemination Cannula



The STUMPY Intra Uterine Insemination Cannula is intended for the introduction of washed spermatozoa into the uterine cavity.

- Rounded tip
- Luer lock
- Flexible cannula
- Small size catheter




Order No.	Description	Qty. / Box
ST-IUI	STUMPY Intra Uterine Insemination Cannula	25

\*All products may not be available in all countries. Please contact our sales department for details.



# Female Reproductive System

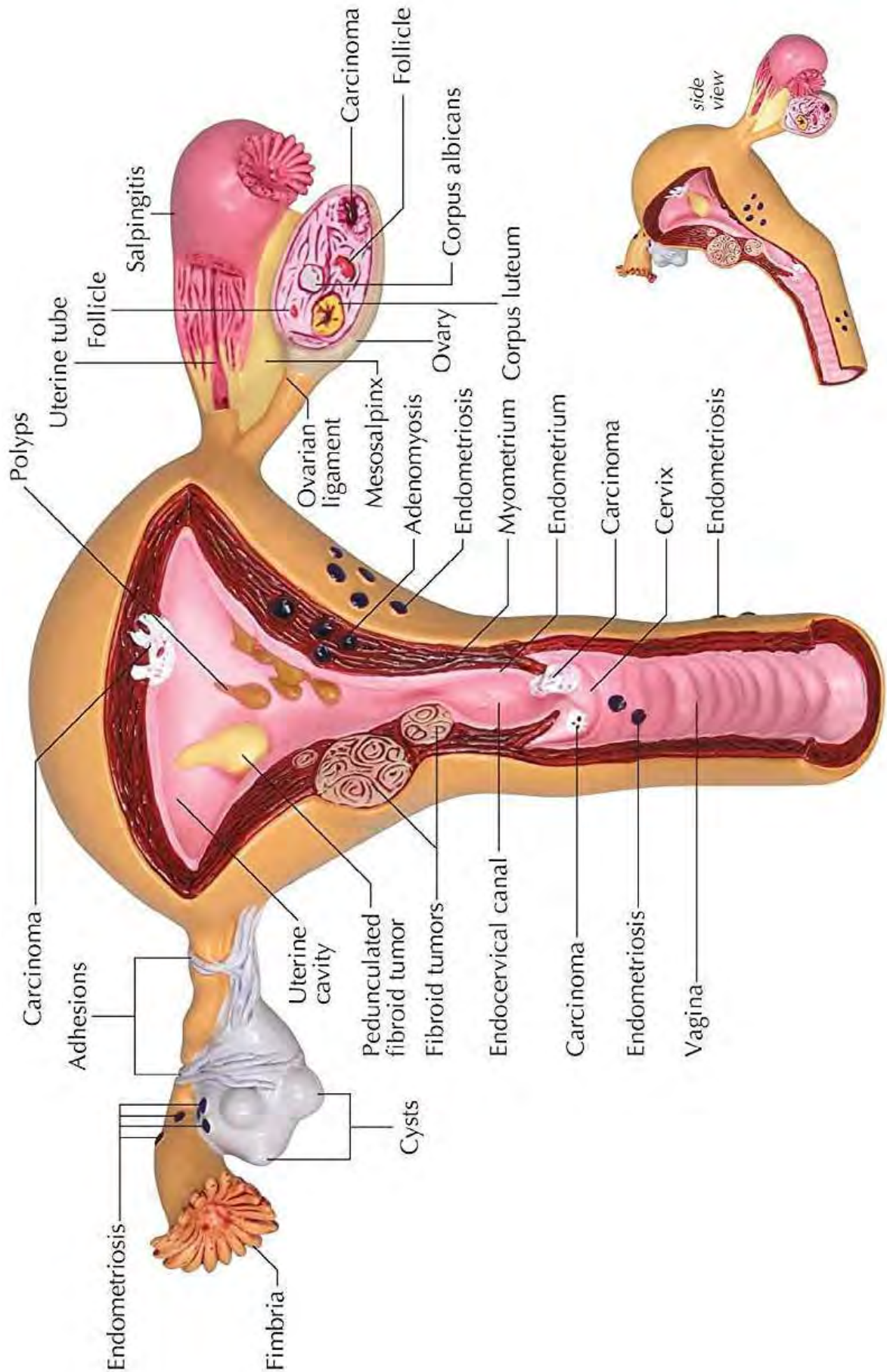



+1 714-572-1709



info@allwinmedical.com



www.allwinmedical.com

